

Title: **Cleaning of Telemetry Modules #7**

Responsibility: RCMS Monitor Technician

Purpose of Guidelines: To clean telemetry modules per manufactures recommendation prior to next patient use.

Procedure:

1. Don gloves.
2. Remove telemetry module from return bin
3. Remove batteries.
4. Use PDI screen wipes (70% alcohol) to wipe out battery compartment.
5. Unplug rubber stopper and clean them and rest of module.
6. If overly saturated leave to dry.
7. Replace telemetry module to appropriate bin/slot.
8. Remove gloves.
9. Wash hands.

Todd Korzec, RN, BSN
Director Cardiovascular Services

Date

/s/

Samer Khouri, MD
Director, Non-Invasive Cardiac Imaging

03/16/2022

Date

/s/

Christine Stesney-Ridenour, FACHE
Chief Operating Officer- UTMC

03/16/2022

Date

/s/

03/16/2022