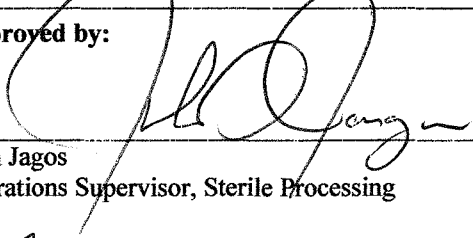
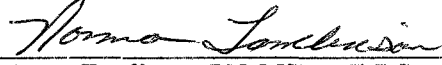


11. If a true positive result is observed repeat the biological test procedure on a second vial that has been processed in the sterilizer. If a positive result is again obtained, have the sterilizer checked by Bio-med personnel.
12. Read and Record results at 24hrs.
13. Use one control vial of the same Lot # per day for two sterilizers.
14. Scan the cycle printout and enter Biological test information and results in the computerized sterilization management system in accordance with policy # 3364-139-06-01, Sterilization Records in Sterile Processing.

<p>Approved by:</p> <div style="text-align: center;">  _____ John Jagos Operations Supervisor, Sterile Processing </div> <div style="text-align: center; margin-top: 20px;">  _____ Norma Tomlinson, RN, MSN, NE-BC Associate Vice President Associate Executive Director </div> <p><i>Review/Revision Completed By:</i> <i>Operations Supervisor, Sterile Processing</i></p>	<p>Review/Revision Date:</p> <p>1996 1998 1999 2002 2005 2007 1/2009</p> <hr/> <p>Next Review Date: January 22, 2012</p>
<p>Policies Superseded by This Policy: SP-6-21 Biological Spore Test for Hydrogen Peroxide Plasma (Sterrad 100 Sterilizer)</p>	

It is the responsibility of the reader to verify with the responsible agent that this is the most current version of the policy.