

<p>Name of Policy: Weight Limits of Imaging Tables</p> <p>Policy Number: 3364-135-001</p> <p>Approving Officer: Assistant Professor & Deputy Clinical Service Chief, Director, Radiology</p> <p>Responsible Agent: Assistant Professor & Deputy Clinical Service Chief, Director, Radiology</p> <p>Scope: University of Toledo Medical Center Radiology</p>	 <p>Effective date:</p> <p>Original effective date: 06/01/2017</p>
Key words: Weight limits, Safe Operation, Imaging Tables, Manufacturer, Patient Safety	
<input type="checkbox"/> New policy proposal	<input checked="" type="checkbox"/> Minor/technical revision of existing policy
<input type="checkbox"/> Major revision of existing policy	<input type="checkbox"/> Reaffirmation of existing policy

(A) Policy statement

To ensure no patient is placed on the Imaging Tables that exceeds the manufacturer's weight limit for safe operation.

(B) Purpose of policy

To comply with the Safe Medical Device Act of 1990 and provide a safe procedural environment for all patients of the Cardiovascular/DP Lab.

(C) Scope

1. The Table weight limits are as follows:
 - a. GE Discovery XR650 elevating table 220kg (485 lbs)
 - b. GE Diffinium 8000 elevating table 200 kg (485 lbs)
 - c. GE RF P180 Footboard and Table with tabletop movement = 585 lbs (266 kg)
 - Table without tabletop movement = 710 lbs (323 kg)
 - d. Philips MD Eleva 185 kg (407 lbs)
 - e. GE Prodigy 350 lbs (dexa)
 - f. Toshiba Aquilion One, 320 300 kg (660 lbs)
 - g. Toshiba Aquilion 64 204 kg (450 lbs) Abdominal exams 227 kg (500) Head exams
 - h. GE Premier 3T – 250 kg (550) lbs
 - i. [GE Signa Artist 1.5T - Siemens Espree 1.5T - 200227 kg \(440500 lbs\)](#)
 - j. GE Infinia Hawkeye 182 kg (400 lbs)
 - k. Philips Gemini 64 227 kg (500 lbs)
 - k.l. [GE Definium 656 HD – 320 kg \(705 lbs\)](#)
2. Patient weight will be obtained from the patient's record. If patients' weight is within 15 kg (30 lbs.) of the table limit:

← **Formatted:** Tab stops: 0.88", Left + 1", Left

- a. Patient will be weighed prior to scheduled procedure for a real time accurate weight.
- b. If patient found to be under the weight limit of the table, the procedure will continue as scheduled.
- c. If patient is found to be over the weight limit the procedure physician will be notified and the procedure will be rescheduled or cancelled.

Approved by:	Policies Superseded by This Policy: • <i>None</i>
<hr/> Nathan Egbert, MD Assistant Professor & Deputy Clinical Service Chief	Initial effective date: 06/01/2017
<hr/> Date	Review/Revision Date: 06/01/2017 06/01/2020 01/01/2023 01/01/2025
<hr/> Ryan Landis, BSRT (R)(CT) Director, Radiology	
<hr/> Date	
<hr/> Daniel Barbee Chief Executive Officer UTMC	
<hr/> Date	Next review date:
<i>Review/Revision Completed by:</i>	
Ryan Landis, BSRT (R)(CT)	