

**Guideline:** ECG Cleaning

**Guideline Number:** 1

**Responsible Department:** All Hospital Departments that use ECG Machines



**Effective date:** 1/2025

**Scope:** University of Toledo Medical Center

**Procedure:**

After patient use:

1. Don gloves
2. Wipe wires and MAC module with wipes that contain Sodium Hypochlorite 5.2% minimum 1:500 dilution. Sani-Cloth AF3 may be used.
3. Wait 3 minutes.
4. After three minutes may wipe with a clean damp cloth to prevent residual.

Monthly Cleaning:

1. Don gloves
2. Wipe wires, MAC module, keyboard, case, and base with wipes that contain Sodium Hypochlorite 5.2% minimum 1:500 dilution. Sani-Cloth AF3 Wipes may be used.
3. Wait 3 minutes.
4. After three minutes may wipe with a clean damp cloth to prevent residual.

## Cleaning ECG Machine Guideline

Reviewed by:

/s/

---

Name: Todd Korzec RN BSN  
Title: Director of Cardiovascular  
Services

1/2025

---

Date

*Review/Revision Completed by: Todd  
Korzec RN BSN*

Initial effective date:  
1/2020

Review/Revision Date:  
1/2025

Next review date: 1/2028