

**Guideline:** Cleaning of Telemetry Modules

**Guideline Number:** RCMS 7

**Responsible Department:** Remote Cardiac Monitoring Station



**Effective date:** 2/2025

**Scope:** University of Toledo Medical Center

**Procedure:**

1. Don gloves.
2. Remove telemetry module from return bin
3. Remove batteries.
4. Use PDI screen wipes (70% alcohol) to wipe out battery compartment.
5. Unplug rubber stopper and clean them and rest of module.
6. If overly saturated leave to dry.
7. Replace telemetry module to appropriate bin/slot.
8. Remove gloves.
9. Wash hands.

Reviewed by:  
/s/

Initial effective date:  
1/2020

---

Name: Todd Korzec RN BSN  
Title: Director of Cardiovascular  
Services

Review/Revision Date:  
3/16/2022  
2/2025

2/26/2025

Next review date: 2/2028

---

Date

*Review/Revision Completed by: Todd  
Korzec RN BSN*

