Name of Policy:Rejection CriteriaPolicy Number:3364-136-CBGL-05		UT UTOLEDO HEALTH	
Approving Officer: Medical Director, Blog Program	Effective date: 2/3/2025		
Responsible Agent : Director, Respiratory C Services	Original effective date : 4/2003		
Scope : The University of Toledo Medical C Respiratory Care Services Department			
Key words: Rejection Criteria, Blood Gas, Lab, Specimen, labeling			
New policy proposal		Minor/technical revision of existing policy	
Major revision of existing policy		Reaffirmation of existing policy	

(A) Policy statement

The Blood Gas Lab will have a policy for recognition and handling of specimens unacceptable for analysis.

(B) Purpose of policy

To establish procedures for specimens rejected by the blood gas lab.

- (C) Procedure
- 1. If a specimen is received into the blood gas lab with a needle still attached, without a label, or if the specimen is miss-labeled, the specimen will be rejected. A specimen is considered miss-labeled if there is incorrect patient information on the label. The nursing unit should be notified when the specimen is rejected. The Respiratory Care Practitioner (RCP) will cancel the original order and reorder a new blood gas in the electronic medical record (EMR) once a new and appropriate sample is obtained.
- 2. The blood gas sample should arrive in an ice bath. Without ice, the sample must be analyzed within 15 minutes for arterial samples and 30 minutes for venous samples (in a proper lab tube) Samples received without these accommodations will be rejected, and the nursing unit notified.
- 3. If a specimen is determined unacceptable due to clotting or an insufficient amount of blood, the sample will be rejected, and an attempt to collect a sufficient sample is needed.

Approved by:	Policies Superseded by This Policy:
	• <i>n/a</i>
/s/	
	Initial effective date: 04/2003
Melissa Kukiela BSRC, RRT	
Director, Respiratory Care Services	Review/Revision Date:
	12/15/2004
2/3/2025	11/17/2005
	12/06/2006
Date	04/17/2007
Date	03/01//2008
	02/12/2009
/s/	04/29/2010 02/23/2011
	02/23/2011 04/11/2012
Lauren Stanoszek, MD	03/03/2014
Medical Director	05/20/2014
	03/01/2017
2/3/2025	02/12/2019
	09/21/2020
Date	03/15/2023
Date	02/03/2025
Review/Revision Completed by:	Next review date: 02/3/2027
Director, Respiratory Care Services	1 VOAT I C VIC W UALC. 02/3/2027